



کتر شستشوی رحم و

انتقال جنین اسب

مشخصات :

طراحی و ساخت کتر شستشوی رحم و انتقال جنین اسب به نحوی است که ایمنی کاربرد آن را کاملاً تأمین می نماید . ضمناً کاربرد کتر باعث خراشیدگی مجرای ورود و محوطه واژن و رحم نمی شود .

دستور العمل مصرف :

*این وسیله در دامپزشکی جهت انتقال جنین اسب و شستشوی رحم و تخلیه محتویات در اقدامات مامایی و درمان کاربرد دارد .

۱. پس از آماده نمودن مادیان و خارج کردن کتر از بسته بندی ، متخصص مامایی دامپزشکی از طریق واژن قسمت کافدار کتر را با شرایط استریل و بهداشتی به سرویکس (گردن رحم) هدایت می نماید .

۲. پس از عبور کتر از سرویکس و ورود به محوطه رحمی با سرنگ هوا بسته داخل کاف دمیده می شود و با تأیید پر شدن کاف با هوای کافی توسط تست کاف ، دریچه خروج هوا بلا فاصله بسته می شود .

۳. ضمن اطمینان از ثابت شدن کاف ، مایعات از طریق انتهای لوله وارد محوطه رحمی می شود و پس از اتمام کار و پایان شستشو از همان طریق مایعات تخلیه می شود .

*در صورت آسیب دیدن بسته بندی مورد استفاده قرار نگیرد .

* پس از مصرف از بین برده شود .

هشدار: در صورت استفاده مجدد خطر آلودگی میکروبی وجود دارد.

EQUINE CATHETER

Specification :

This product consists of a tube with connector , test cuff , supply , cuff and 2 openings.

Application :

This device is used for embryo transferring , cleaning and washing of the genital tract of the mare. The advantage of this application is the excellent uterine drainage.

Instructions For Use:

- 1.After preparation of the mare , remove the catheter and insert it into vagina under sterile and aseptic conditions.
2. Pass the catheter from cervix and insert it into uterine cavity and inject air into cuff. Confirm adequacy of air injection by use of test cuff and then close the air supply route.
- 3.Ensure the cuff is fixed and then inject the fluids into uterine cavity and perform the desired procedure.

*Do not use if package is damaged.

*Destroy after use.

Warning : If you re-use it will be in microbe pollution danger.

258-S5-146/11



MADE IN IRAN

Manufacturer : SUPA MEDICAL DEVICES
MANUFACTURE SITE: 8 th km of Karaj-Qazvin
High way, Karaj, Iran

Postal code: 3365166141

TEL (CUSTOMER SERVICES) : +98 21 44422991
TEL (Central Office) : +98 21 4442283

Fax: +98 21 44401430
E-mail : info@supamedical.com