(PVDF) SUTURE

Sterile Synthetic Non-Absorbable Suture

Poly Vinylidene Fluoride monofilament (PVDF) is a synthetic, nonabsorbable suture and it has minimal tissue reaction with the wound edge. One of the advantages of this suture is its secure knot and it is inert and does not provide harbor for bacterial growth. It is remarkably strong and considerably less thrombogenic than much other suture materials. One of the benefits of these sutures is the wide range of needles. Color is dark blue.

Application:

PVDF is indicated for use in vascular, muscle and subcuticular tissue approximation and/or ligation. Suture should be selected and used depending on patient's conditions, surgical experience, surgical technique and wound size.

Advantages:

PVDF has greater pliability for easier handling and tying. In addition it has high tensile strength. It has less packing memory and can be handled more easily.

Adverse Reactions:

Adverse reactions associated with the use of this device include, allergic response, in certain patients and transient local irritation at the wound site followed by moderate and transient inflammation due to foreign body response. Like all foreign bodies PVDF may enhance an existing infection.

Caution:

Care should be taken to avoid damaging the surface of PVDF suture with surgical instruments in use as this could lead to fracture of the material. The needle holder should be carefully selected to match the size and strength of the needle to be used. Needle should be grasped securely in the tip of the needle holder jaws approximately one third of the total needle length from the butt.

PVDF sutures are sterilized by E.T.O.

Do not re-sterilize.

*Do not use if package is damaged.

*Destroy not used suture.

Storage:

Recommended storage conditions: keep away from of the sunlight & temperature. Do not use after expiry date.

Warning: If you re-use it will be in microbe pollution danger.

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