

کاتتر کات دان



راهنمای مصرف:

ابتدا محل مورد نظر را با بتادین یا مانند آن ضدعفونی نموده پس از آن با استفاده از زایلوکائین یا مانند آن بیحسی موضعی ایجاد نموده و در شرایط استریل پس از یافتن ورید و ایجاد سوراخ در آن ، کاتتر را وارد رگ نموده و سپس محل را بخیه زده و پانسمان نمایید.

* در صورت آسیب دیدن بسته بندی مورد استفاده قرار نگیرد.

* پس از مصرف از بین برده شود .

هشدار: در صورت استفاده مجدد خطر آلودگی میکروبی وجود

دارد.

Cut down catheter

Instructions for use:

At the beginning, the intended site should be disinfected by using povidoneiodine and so on .Then local anaesthesia is made with xylocaine or so on , and within sterilized conditions, after finding the vein and making a pore, catheter is inserted in it.

Then the site is sutured and dressed.

***Do not use if package is damaged.**

***Destroy after use.**

Warning : If you re-use it will be in microbe pollution danger.



258-SS-037/15

Manufacturer : SUPA MEDICAL DEVICES

MANUFACTURE SITE: 3 in bin of Karaj- Qom

High way: Karaj-Iran

Postal code: 335156141

TEL (CUSTOMER SERVICE): +98 21 4422381

TEL (General Office): + 98 21 4422383

E-mail: info@supamedical.com