Blood and infusion warmer extension set

Application :

This set is used during administration of blood, blood derivatives and other infusion products (for example during or after surgery). Blood (Plasma,...) is warmed by blood warmer machine, when flow in the tube of set.

Instructions for use:

1. Prepare the blood bag and its connections.
2. Wind the tube around blood warmer machine along specially designed and patented groove space.
3. Open roller clamp, expel the air by letting blood (plasma,...) flow in the tube and close roller clamp.
4. Connect outlet connector to IV catheter (or another device which keeps vein open) and has already been fixed in place.
5. After blood (plasma,...) is warmed up to appropriate temp. (37 – 40 ° C), open roller clamp and let blood enter vein via IV catheter.
6. After transfusion of proper amount close roller clamp, and disconnect end of the set from IV catheter.

Note : There is an asymmetric Y shape connector segment for administration of other drugs .

*Do not use if package is damaged .

*Destroy after use.

Warning : If you re-use it will be in microbe pollution danger.

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Manufacturer : SUPA MEDICAL DEVICES
Manufacture Site: 8th km of Karaj-Qazvin
High way, Karaj, Iran.
Postal code: 3365166141
TEL(CUSTOMER SERVICES) : +98 21 44422991
TEL (Sales Department): + 98 21 4442283
Fax : + 98 21 44401430
E-mail : info@supamedical.com

258-S5-093/14