



## Protection Cover of

### Artificial Insemination Sheath

This device is used in insemination procedures in domestic animals to prevent dissemination of vaginal infection into uterus.

#### Characteristics:

This product is designed and produced in such a manner to prevent scratching of vagina and cervix and therefore it prevents contamination of deep genital tract (cervix and uterus) by vaginal infections.

#### Instructions for use:

1-Open the package of product and insert the artificial insemination sheath into the protector cover and advance the sheath to distal end (near the sheath cap) of protector cover.

2- Insert the sheath and the protector into vagina and place it just behind the cervix, so that the cap is placed at the distal end of the sheath.

3- Ensure that the sheath is out of the cap and introduce the sperm content into uterus.

\*Do not use if package is damaged.

\*Destroy after use.

Warning : If you re-use it will be in microbe pollution danger.



MADE IN IRAN



## روکش (محافظ)

### غلاف تلقیح مصنوعی

این وسیله در عملیات تلقیح در دامپروری ها برای جلوگیری از انتقال عفونت محوطه واژنی به رحم به کار می رود .

طراحی و ساخت این محصول به گونه ای است که باعث جلوگیری از خراشیدگی محوطه واژن و گردن رحم شده و از ورود و انتقال عفونت محوطه واژنی به نواحی عمقی تناسلی (سرویکس و رحم) جلوگیری می شود.

#### روش کاربرد:

۱- پس از خارج کردن محصول از بسته بندی ، با قرار دادن غلاف تلقیح مصنوعی در داخل محافظ ، غلاف تا قسمت انتهایی (نزدیک کلاهک روکش ) محافظ هدایت می شود.

۲- پس از عبور غلاف تلقیح مصنوعی و محافظ از واژن و قرار گرفتن در پشت سرویکس غلاف از قسمت انتهایی کلاهک محافظ خارج می شود.

۳- با اطمینان از خروج غلاف از انتهای کلاهک محافظ تخلیه اسپرم انجام می شود.

\* در صورت آسیب دیدن بسته بندی مورد استفاده قرار نگیرد .

\* پس از مصرف از بین برده شود .

هشدار: در صورت استفاده مجدد خطر آلودگی میکروبی وجود دارد.



Manufacturer : SUPA MEDICAL DEVICES

MANUFACTURE SITE: 8 th km of Karaj-Qazvin High way, Karaj, Iran

Postal code: 3365166141

TEL ( CUSTOMER SERVICES ) : +98 21 44422991

TEL (Central Office): + 98 21 44422283

TEL (Sales Department): + 98 21 44469017

E-mail : [info@supamedical.com](mailto:info@supamedical.com)

258-S5-109\11