



ARTIFICIAL INSEMINATION SHEATH

Indication:

This device is used in animal husbandry as a protective cover for artificial insemination syringe containing sperm.

Instructions For Use:

Place syringe inside artificial insemination pipette and insert into the cervix through the vaginal canal under aseptic conditions. Eject sperm after proper insertion.

***Do not use if package is damaged.**

***Destroy after use.**

Warning: if you re-use it will be in microbe pollution danger.

غلاف تلقیح مصنوعی



کاربرد :

این وسیله در دامپروریها به عنوان روکش روپسرنگ تلقیح مصنوعی حاوی پسایت اسپرم قرار می گیرد . کانکتور در ابتدای غلاف برای استفاده از پایت های ۱/۴ و ۱/۸ سی سی است.

طریقه مصرف :

پس از قرار دادن سرنگ تلقیح مصنوعی حاوی پایت در داخل غلاف تلقیح مصنوعی ، آن را با توجه به رعایت نکات بهداشتی از واژن و سرویکس (گردن رحم) عبور داده و توسط سرنگ تلقیح اقدام به تخلیه اسپرم می کنند.

*در صورت آسیب دیدن بسته بندی مورد استفاده قرار نگیرد.
* پس از مصرف از بین برده شود.

هشدار: در صورت استفاده مجدد خطر آلودگی میکروبی وجود دارد.



MADE IN IRAN



Manufacturer : SUPA MEDICAL DEVICES

Manufacture Site: 8th km of Karaj-Qazvin

High way, Karaj, Iran.

Postal code: 3365166141

TEL (CUSTOMER SERVICES) : +98 21 44422991

TEL (Sales Department): + 98 21 44469017

TEL (Central Office): + 98 21 44422283

E-mail : info@supamedical.com

258-S5-107/9