Foley Catheter 2 way siliconised

This device is used for discharging bladder for extended periods and is secured inside the bladder by inflating the balloon.

Instructions for use:

1. Lay patient in specific position and make catheterizing set ready.
2. Remove Foley catheter out of outer package and put it on sterile drape.
3. Wash the sites with sterile gloves on, bring catheter tip out of inner package.
4. Smear Foley catheter's tip in gel and insert it in urinary meatus. Usually, after one-third of catheter is inserted in female urethra, and about one-half of it in male urethra, small amount of urine flows to the rest of inner package of catheter and shows that it is inside the bladder.
5. Foley's catheter is inserted into bladder up to 4cm left out side.
6. Fill balloon with serum or distilled water according to required amount, which is printed on the device .
7. Pull catheter to fix balloon inside bladder, then connect catheter to urine bag and fix the set by sticking to patient's leg.

*Do not use if package is damaged.

*Destroy after use.

Warning : If you re-use it will be in microbe pollution danger.

Notice : This catheter is siliconised. so this is hyposensitive and you can use it easily.

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Manufacturer : SUPA MEDICAL DEVICES

Manufacture Site: 8th km of Karaj-Qazvin

High way, Karaj, Iran.

Postal Code : 3365166141

TEL(CUSTOMER SERVICES) : +98 21 44422991

TEL (Sales Department): + 98 21 44469017

TEL (Central office) : + 98 21 44422283

E-mail : info@supamedical.com

258-S5-218/1